

imp De Mayo D² Knee Positioner® Carriage Instructions For Use



Carriage Removal



1. Remove the E-Brake from the carriage.

2. Pull the handle towards you and insert a flat-head screwdriver into the screw on the handle.



3. Create a 4 to 6 millimeter gap between the L plate and the carriage.




imp De Mayo D² Knee Positioner® Carriage Instructions For Use



4. Slide the carriage towards the end of the Base Plate aligned with the CE mark.


5. Slide the carriage around the curved end of the Top Plate.

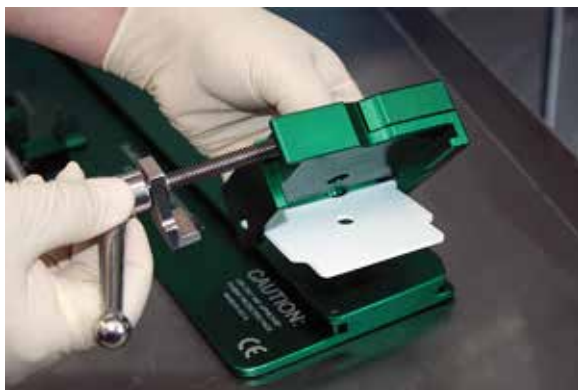

6. Unthread the handle, remove it, and open the carriage for cleaning.



Carriage Replacement

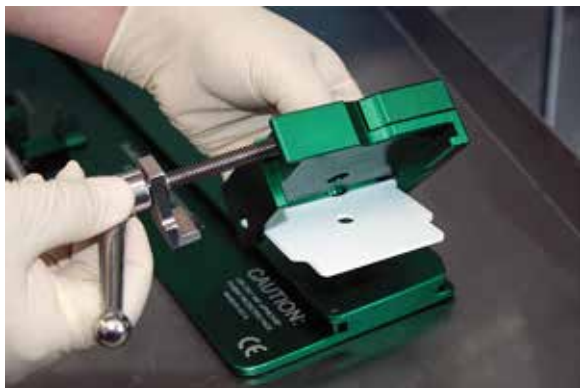

1. Close the plates of the carriage and thread the handle back into the carriage leaving a 4 to 6 millimeter gap between the L plate and the carriage.




2. While holding the Teflon Pad, slide the carriage onto the top plate starting on the handle side, then around the curved end on the opposite side. Make sure the foot of the L plate is in the groove of the Base Plate.



3. Using a flat-head screwdriver, pull the handle towards you and tighten the L plate, leaving enough space to insert the boot and finish tightening the handle.



imp De Mayo D² Knee Positioner® Carriage Instructions For Use

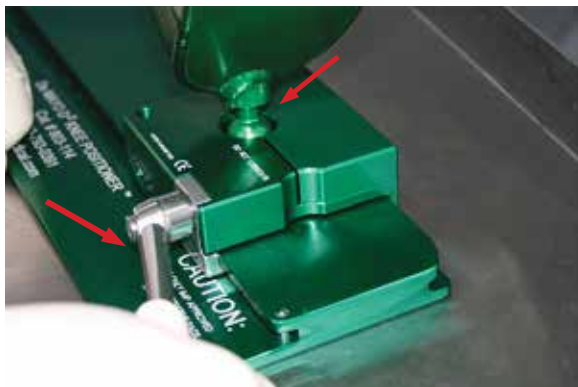

4. With the boot locked into place, pull the handle and rotate to the right to the 3 o'clock position and replace the E-Brake.



Set Up Test Procedure



1 ~ Check that the locking handle locks the ball of the boot and is in the 3 o'clock position.

2 ~ Replace the E-Brake

