


# *imp* De Mayo Push Button™ Clamp


## Instructions For Use

De Mayo Push Button™ Clamp


Cat.# 713-302

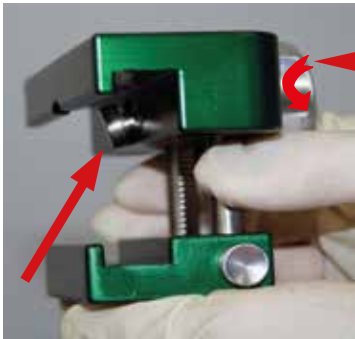


1. Loosen the Clamp Knob all the way by turning it counter clockwise. The leading “cap” should be flush to the upper jaw.

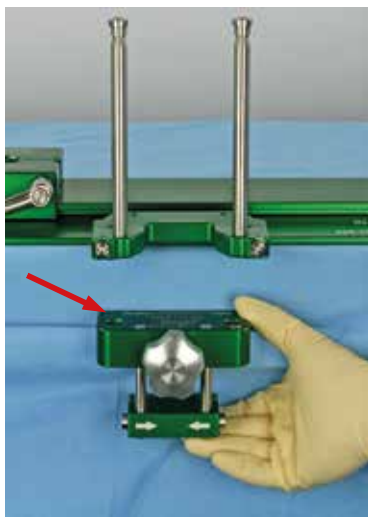

2. Place the Clamp over the drapes with the Clamp holes under sliding pins and “squeeze” the jaws tight on the rail.

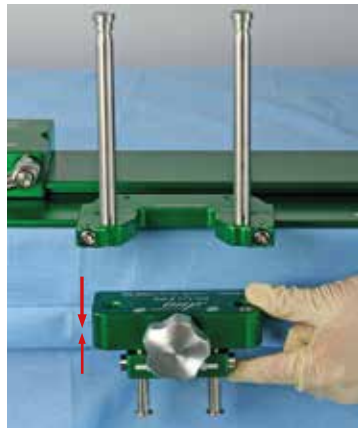


3. Tighten the Clamp Knob by turning it clockwise.

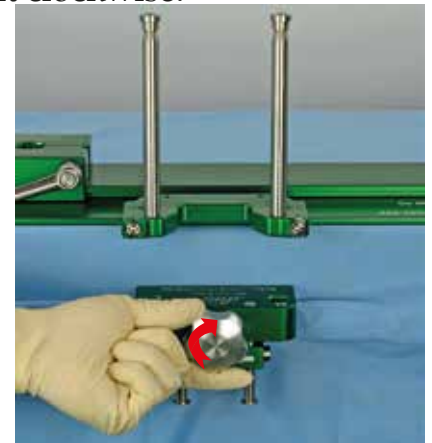


~ locate the Clamp holes under the sliding pins.

4. Seat the sliding pins in the Clamp holes.

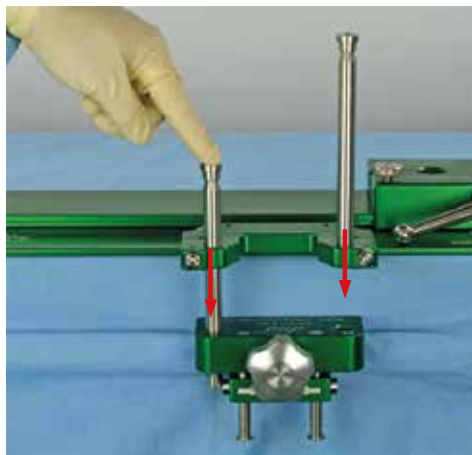

# imp De Mayo Push Button™ Clamp


## Instructions For Use

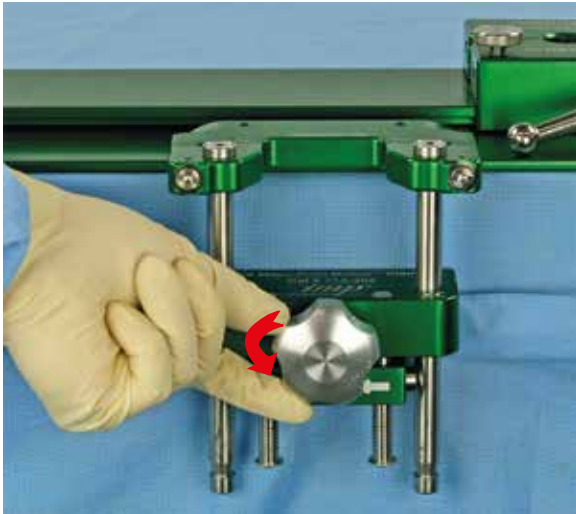


### 1. *To REMOVE:*

Turn the Clamp Knob counter clockwise until it stops.

### 2. Pull the sliding pins out of the Clamp.

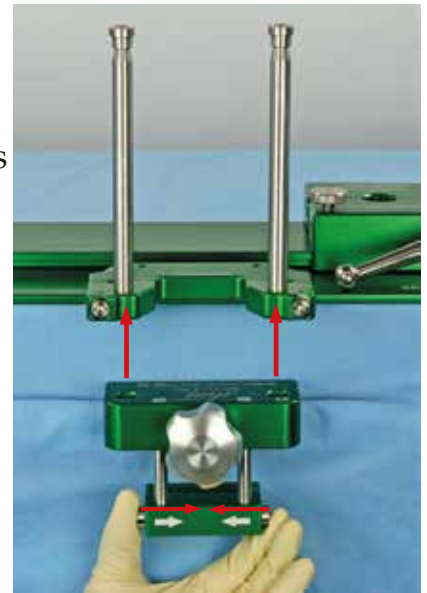

### 3. Lower the bottom jaw by squeezing the release buttons. Remove Clamp.

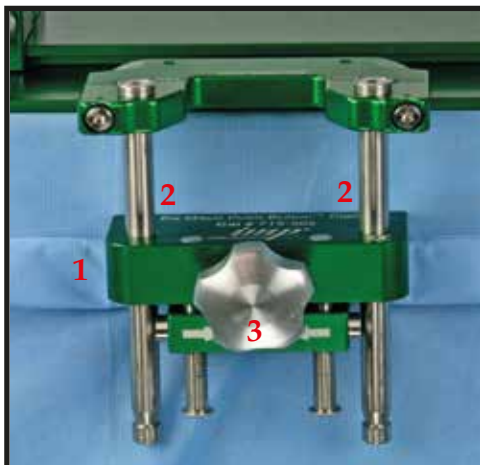

## Set Up Test Procedure



TIGHTEN CLOCKWISE TO LOCK THE KNOBS:

- 1 ~ Check that Clamp is locked on the side rail
- 2 ~ Check that both sliding pins are fully seated
- 3 ~ Check that Clamp Knob is secure

*imp* Innovative

Medical Products, Inc.  
87 Spring Lane, PO Box 8028  
Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805  
Toll Free PH: 1-800-467-4944  
[www.impmedical.com](http://www.impmedical.com)



CE

De Mayo Push Button™ Clamp is a trademark of Innovative Medical Products, Inc.  
De Mayo Push Button™ Clamp - PATENT APPLIED FOR  
© December 2017 IMP  
ALL RIGHTS RESERVED