


De Mayo Ankle Distractor™ System

INSTRUCTIONS FOR USE




1. Drape the leg according to hospital protocol and place Distractor base on the end of the OR table.

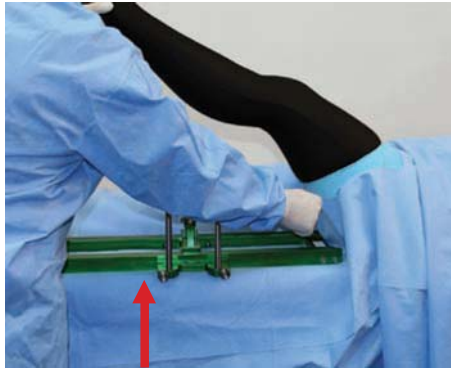


2. Place the sliding "bridge" in the middle of the base.


ATTACHING THE SINGLE LEVER CLAMP

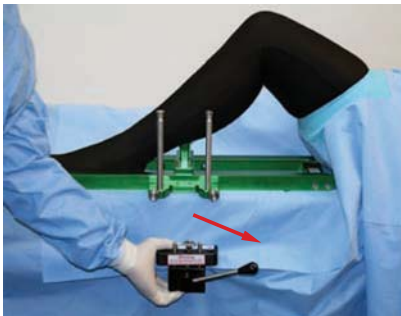


3. Position the clamp under the pins. With Left hand, squeeze the jaws tight and rotate the lever to lock.

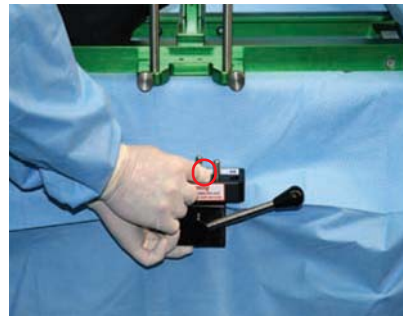


4. Holding the jaws, tighten the knurled knob all the way.

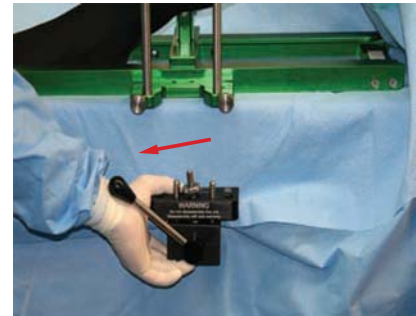


5. While holding the clamp, rotate the lever to unlock.

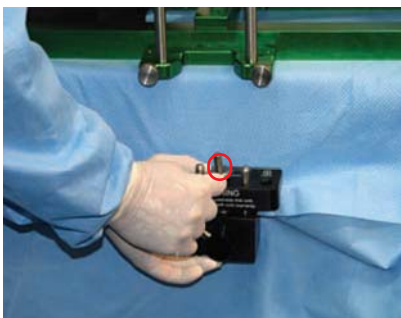


6. Turn the knurled knob 1/4 to 1/2 a rotation.

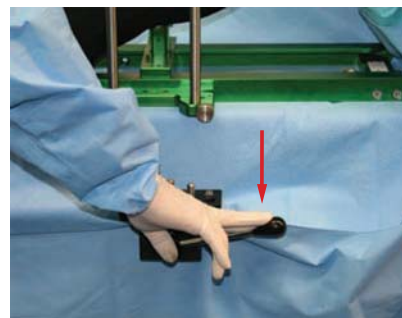

7. Rotate the lever to lock position requiring only 2 finger pressure. Works like a vice grip and "snaps" into place.

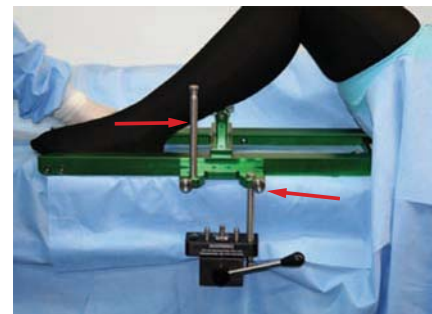

8. Slide pins into clamp.

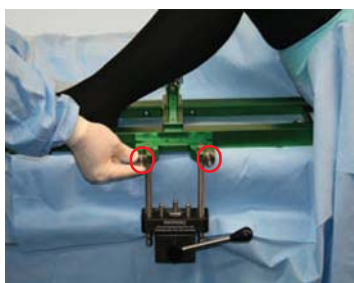

9. Tighten knurled knobs.

De Mayo Ankle Distractor™ System Instructions For Use



10 a. Open the Sterile Patient Protector Pad & Ankle Strap.



10 b. Place on Sterile table.



10 c. Unwrap.



10 d. Remove strap from Pressure Protector Pad.



11. Position the strap on patient's foot with the logo strap on the heel and the dorsal strap on top.



12. Clip the strap to the eye bolt of the base.







13. With the De Mayo Universal Distractor® in the collapsed position,



14. Place the Pressure Protector Pad on the top bar.

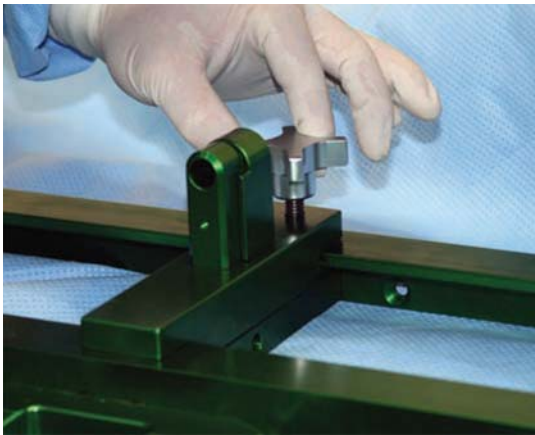

15. With the bridge locked,

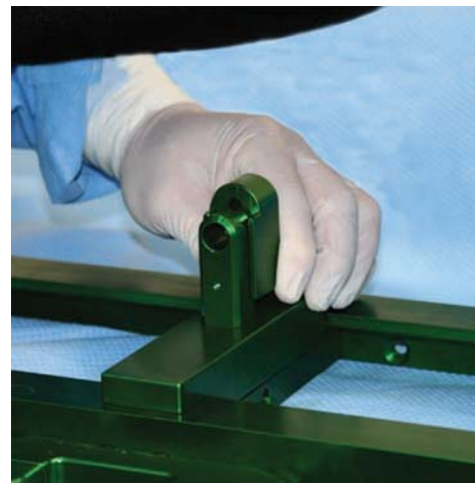


16. Pull up the locking clip.




17. Insert the lower bar of the Distractor and seat the locking clip.




18. Position the Pressure Protector Pad at the distal femur.

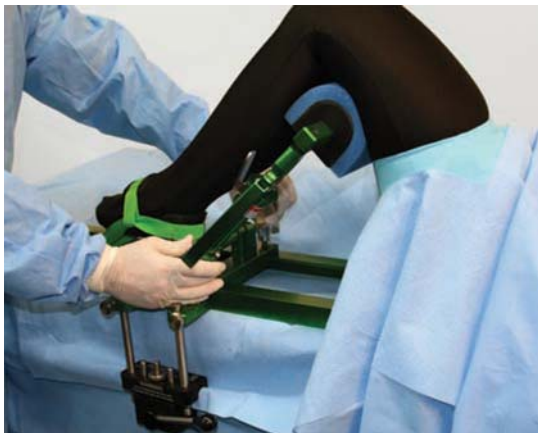

19. Unlock and push the bridge until the strap is tight, and lock in place.

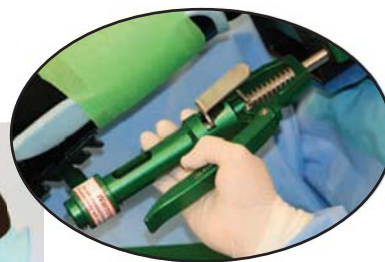

20. Squeeze the De Mayo Universal Distractor® handle for further distraction.

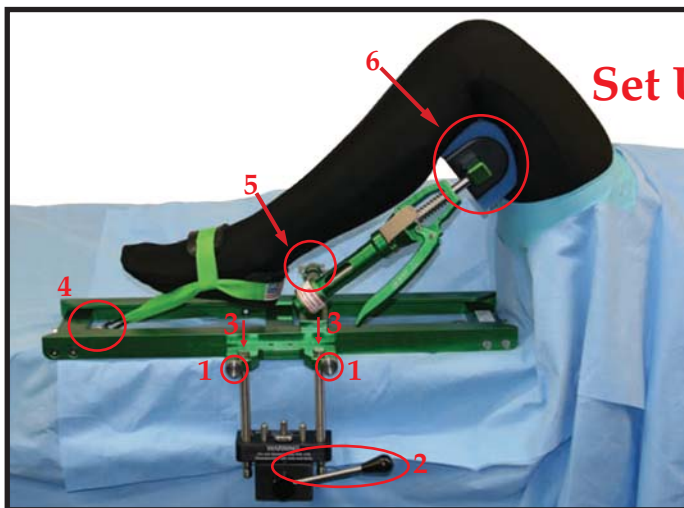

Set Up Test Procedure



Tighten Clockwise to Lock the Knob & Lever

- 1 ~ Lock Both Sliding Pin Knobs
- 2 ~ Lock Lever on Single Lever Clamp
- 3 ~ Check Sliding Pins are Fully Seated
- 4 ~ Ankle Strap Locked to Eye Bolt
- 5 ~ Distractor Bottom Extension is Fully Seated and Locked in Sliding Bridge
- 6 ~ Protective Pad is placed on Top Extension and Properly Centered under Patient's Leg

CE

De Mayo Ankle Distractor™ is a trademark of Innovative Medical Products, Inc.
 De Mayo Ankle Distractor™ PATENT APPLIED FOR
 Single Lever Clamp U.S. Patent No. 7,003,827
 De Mayo Universal Distractor® is a registered trademark of Innovative Medical Products, Inc.
 De Mayo Universal Distractor® U.S. Patent No. 8,048,082
 © January 2014 *ism*®/ Innovative Medical Products, Inc.
 ALL RIGHTS RESERVED

 ***ism*** Innovative
Sports Medicine®

A DIVISION OF INNOVATIVE MEDICAL PRODUCTS, INC.

87 Spring Lane, PO Box 8028
 Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944

www.impmedical.com