


De Mayo Universal Comfort Finger Trap™

Instructions For Use ~ Non-Sterile Technique

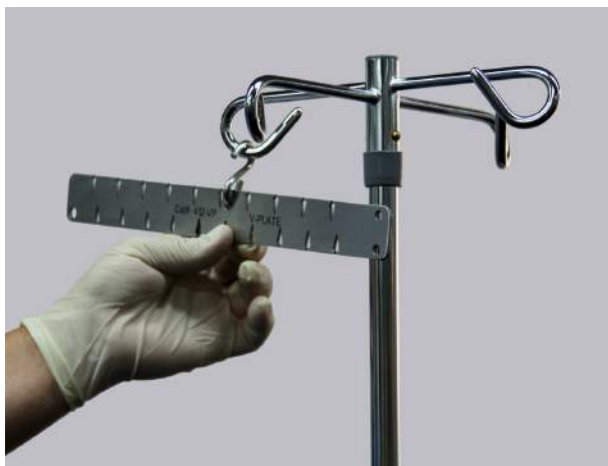

1. Attach "V" Plate to the IV pole.

2. Open Sterile Finger Trap




3. Completely unwrap Velstretch® strap.





4. Fully seat the Finger Trap on patient's thumb.

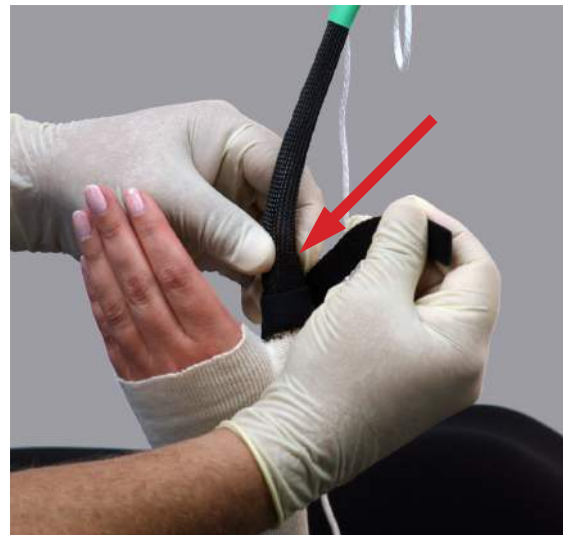


5. Hook-tab must be exposed, not between fingers.




Pull strap around the thumb 2 times, attaching it to the hook material both bottom and top.

6. Seat next trap completely over desired finger. Hook-tab must be exposed, not between fingers.

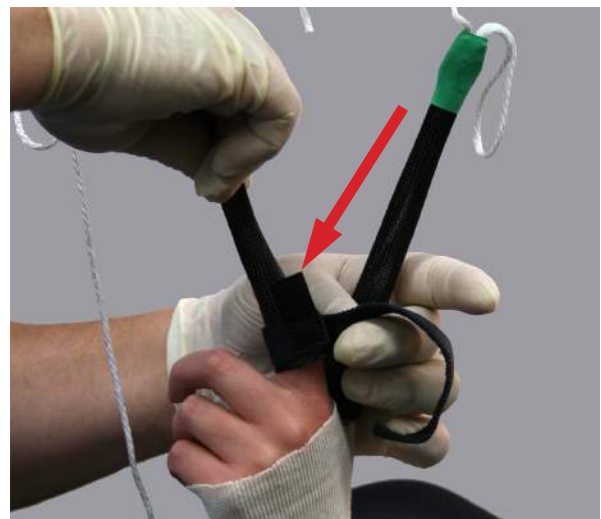




7. Pull strap entirely around first finger, attaching Velstretch® strap to hook material at the base.

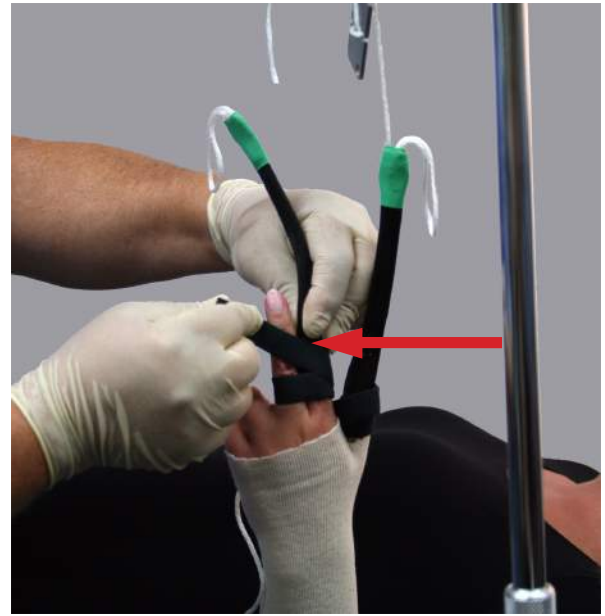

Wrap strap around adjacent finger and attach Velstretch® to top hook material.

Loops

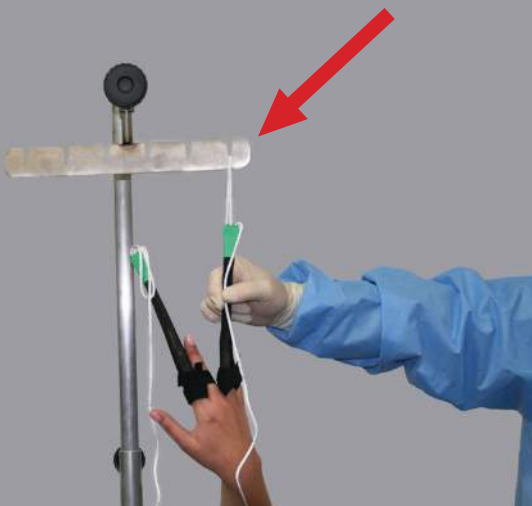


8. Suspend arm by utilizing the attached "Loops"

~ or

for further adjustment utilize the attached "Cords" in "V" Groove.

Cords

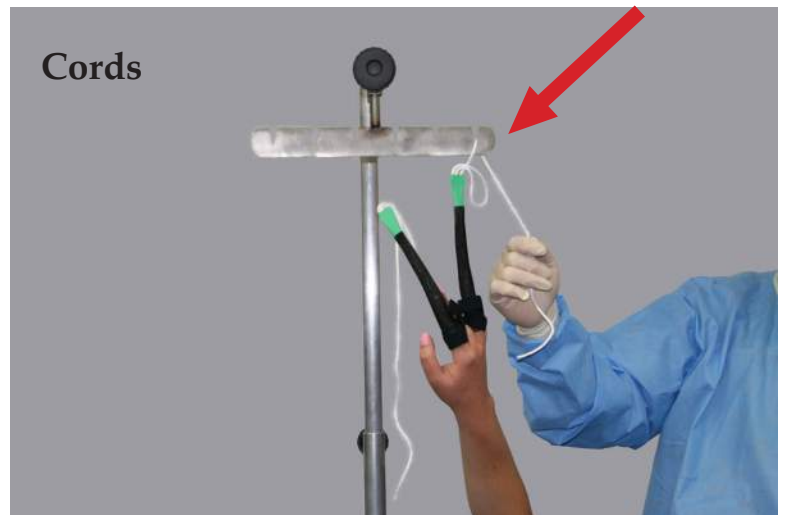

9. Apply casting material according to hospital protocol.



Set Up Test Procedure

1. Check that Velstretch® Strap is securely attached to Hook material.
2. Check that Loop/Cord is securely attached to Finger Trap holder.



imp® **Innovative**
Medical Products, Inc.
87 Spring Lane, PO Box 8028
Plainville, CT 06062
PH: (860) 793-0391 FAX: (866) 459-1805
Toll Free PH: 1-800-467-4944
www.impmedical.com