

Non-Sterile **Phase 4 Gel™ Splint**

Instructions For Use

Sterile Draping Technique Utilizing Hospital Supplied Barriers




1. Three (3) holes allow for different hook placement to control rotation.

← Loop and Hook closures hold splint on patient's arm and provides versatility.

PATENT APPLIED FOR

2. Pre-formed plastic backbone gives conformity to the forearm/wrist anatomic angle.

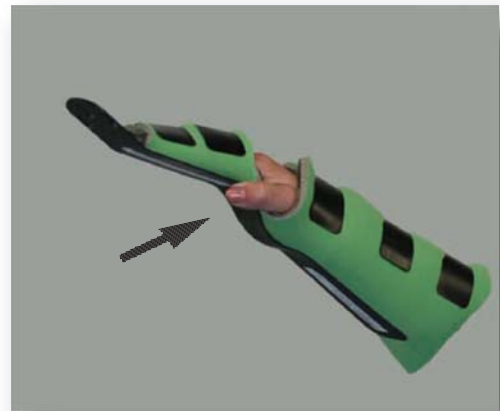

3. Attach to De Mayo RoTractor®

4. Attach hook in applicable location.




Non-Sterile **Phase 4 Gel™ Splint**

Instructions For Use



5. Final step: Apply sterile barrier using either cohesive wrap or other options.

Set Up Test Procedure

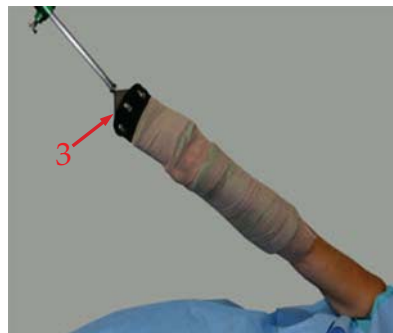

- 1~ Make sure the non-sterile Gel Splint is covered by sterile barrier.
- 2~ Check Hook attachment.
- 3~ Check RoTractor® attachment.



Phase 4 Gel™ is a trademark of Innovative Medical Products, Inc.
 Phase 4 Gel™ Splint - PATENT APPLIED FOR
 De Mayo RoTractor® is a registered trademark of Innovative Medical Products, Inc.
 © Nov 2013 Innovative Medical Products, Inc.
 ALL RIGHTS RESERVED



A DIVISION OF INNOVATIVE MEDICAL PRODUCTS, INC.
 87 Spring Lane, PO Box 8028
 Plainville, CT 06062
 PH: (860) 793-0391 FAX: (866) 459-1805
 Toll Free PH: 1-800-467-4944
www.impmedical.com