


# Reznik Universal Shoulder Positioner™

## Instructions For Use

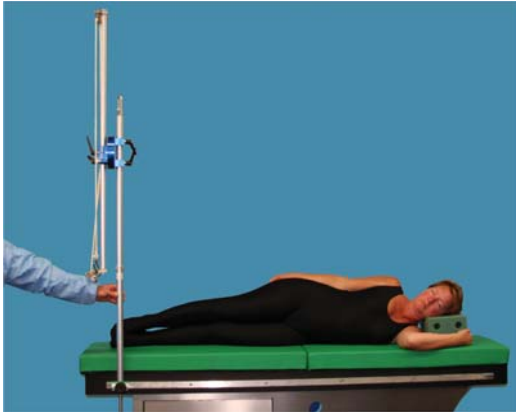


1. Position Rail Clamp and Positioner on the table side where the patient is facing.



2. Insert Positioner into Rail Clamp and rotate Positioner so the Large Lever No. 1 is facing the operator.


Release Lever No. 1 to rotate horizontal bar.

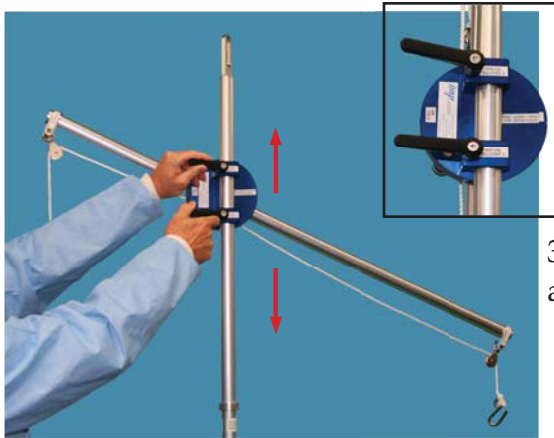

3. Release both Vertical Adjusting Levers No. 2 to achieve appropriate height of Horizontal Bar.



4. Rotate Horizontal Bar so  
a. Head Swivel Pulley faces patient's head,  
b. Traction Swivel faces foot of table

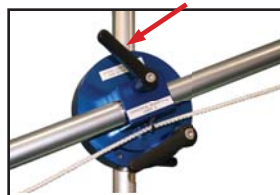


5. Release Horizontal Adjustment Lever No. 3 for horizontal length adjustment.



Optional Feature: De Mayo RoTractor®



Insert RoTractor into Adapter, rotate RoTractor towards patient's head.

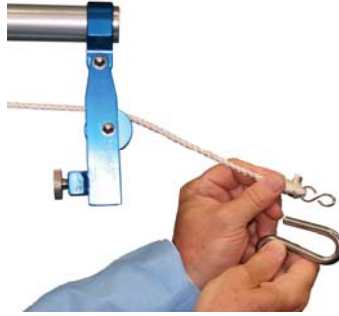

Replace large hook with small hook. Attach hook to eyelet of RoTractor®



6. Lock knob on Rail Clamp and check locking of all levers. Lock all knobs and levers securely.



7. After Phase 4 Gel™ Splint is applied to patient's forearm, secure hook to forearm splint.

8. Apply traction weight to opposite hook, per the surgeon's preference.



Set Up Test Procedure



Tighten to Lock Knobs and Levers securely:

- 1~Rail Clamp
- 2~Lever 1
- 3~Lever 2
- 4~Lever 3
- 5~RoTractor adapter Knob
- 6~Check the slide of the RoTractor



A DIVISION OF INNOVATIVE MEDICAL PRODUCTS, INC.

87 Spring Lane, PO Box 8028  
Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944

www.impmedical.com

CE

Reznik Universal Shoulder Positioner™ is a trademark of Innovative Medical Products, Inc.  
Reznik Universal Shoulder Positioner™ - U.S. Patent No. 7,569,024  
De Mayo RoTractor® is a registered trademark of Innovative Medical Products, Inc.  
De Mayo RoTractor® - PATENT APPLIED FOR  
imp® Rail Clamp is a trademark of Innovative Medical Products, Inc.  
Phase 4 Gel™ Splint is a trademark of Innovative Medical Products, Inc.  
Phase 4 Gel™ Splint - PATENT APPLIED FOR  
© Sept 2013 Innovative Medical Products, Inc.  
All Rights Reserved